IVD

OPTOCHIN

Kat. č.: MLT00086

Pro mikrobiologii

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(EN)

Kat. č.: MLT00086

Diagnostic discs OPTOCHIN are intended for simple routine presumptive detection of *Streptococcus pneumoniae*.

Principle:

Test is based on the susceptibility of *Streptococcus pneumoniae* to optochin (ethylhydrocuprein hydrochloride). *Streptococcus pneumoniae* shows after incubation defined zone of inhibition around the impregnated disc, whilst other viridans-group streptococci are resistant to optochin or show a very small zone of inhibition. Disc OPTOCHIN may also be applied on primary culture plate in order to provide rapid indication of presence of large numbers of *Streptococcus pneumoniae* in sample.

A package of the OPTOCHIN contains:

- 100 diagnostic discs for 100 determinations
- working leaflet

Storage, expiration:

Discs OPTOCHIN should be stored in dark in a refrigerator at (+2 to +8)°C until the expire date indicated on the package.

Required material:

- Blood agar plates
- common laboratory equipment (loops, sterile forceps, burner)

Recommended procedure:

• Prepare suspension corresponding to 0,5 McFarland value in saline (0,85% NaCl) from 24 hours old culture. Streak with a swab on a blood agar plate to obtain almost confluent growth.

Alternative procedure: Pour the prepared suspension on the blood agar plate entirely, suck-off the excess of suspenion by pipette and leave the agar plate to dry up for cca 20 min.

- Note: Simultaneously providing of positive control with known strain is recommended.
- Aseptically place disc OPTOCHIN on the surface of the medium.
 - Incubate in an atmosphere containing (5–10)% CO₂ at 37°C for (18–24) h.

Caution:

- For professional use only.
- After use all discs must be autoclaved or incinerated.

Evaluation:

After incubation read the diameters of inhibition zones:

Inhibition zone (mm)	Result
> 14	<i>Streptococcus pneumoniae</i>
0	viridans-group streptococci other than <i>Streptococcus pneumoniae</i>
< 14	<i>Streptococcus pneumoniae</i> must be confirmed with bile solubility test

Standard control strains:

Quality control of diagnostic discs OPTOCHIN may be provided with standard control strains:

Strain	Control
<i>Streptococcus pneumoniae</i> CCM 4501	positive
<i>Streptococcus constellatus</i> CCM 4043	negative

These strains are distributed by:

CCM – Czech Collection of Microorganisms, Masaryk University, Faculty of Science,

Kamenice 5, building A25, 625 00 Brno, CZ

Tel. +420 549 491 430, Fax +420 549 498 289, <http://www.sci.muni.cz/ccm>, e-mail: ccm@sci.muni.cz

Strains are supplied in freeze-dried form or on gelatin discs.

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